***Read the instructions carefully before using this device.***

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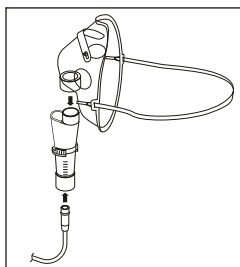
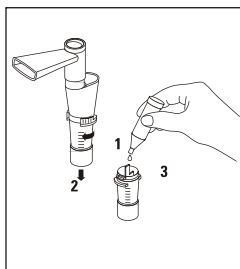
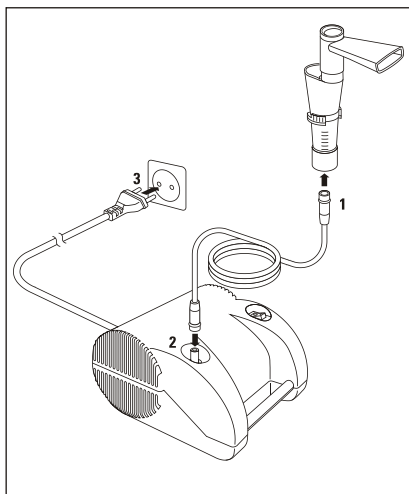
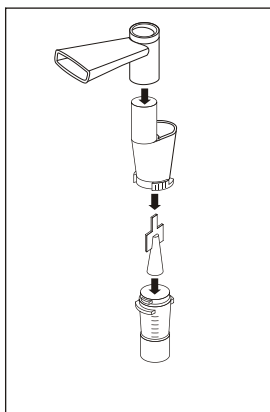
Respironics New Jersey, Inc.  
5 Wood Hollow Road  
Parsippany, NJ 07054 / USA

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#### 1. Introduction

---

#### 2. Preparation and usage of the device

---

#### 3. Cleaning

---

#### 4. Maintenance, care, and service

---

#### 5. Malfunctions and actions to take

---

#### 6. Additional safety instructions

---

#### 7. Guarantee

---

#### 8. Technical specifications

---

##### Device symbols

 ON (power)

 ATTENTION: Read the instructions carefully

 OFF (power)

 Alternating current

 Type B

 Class II

 Separate collection

# 1. Introduction

---

Your new Microlife nebuliser is a high-quality device for inhalation therapy for asthma, chronic bronchitis, and other respiratory illnesses.

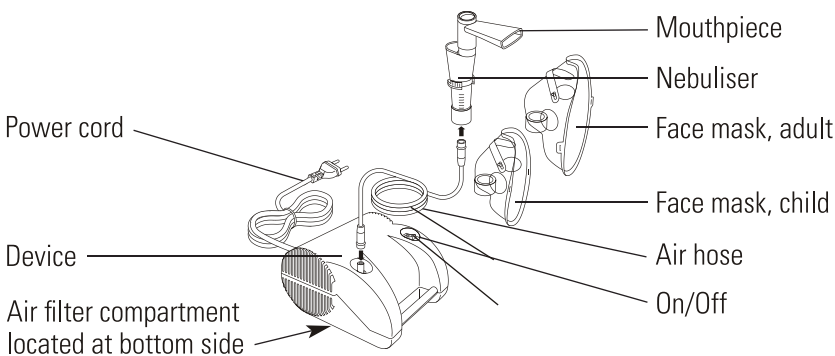
The device is capable of generating a medication mist of particles that are small enough to reach even the deepest regions of your lungs and be of maximal benefit.

Please only use the medication prescribed to you by your doctor and follow your doctor's instructions with regard to dosage, duration, and frequency of the therapy. For safety reasons, the device and all its accessories should only be used by one person.

The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy.

Prior to using the device, please read this manual carefully and retain it in a safe place for future reference.

The figure shows the Microlife inhalation device:

## 2. Preparation and usage of the device

---

Prior to using the device for the first time, we recommend cleaning it as per the instructions in this manual.

Please refer to the foldout page with the figures on the correct usage of the device at the beginning of the manual.

1. Please assemble the nebuliser as shown in the figures. Ensure that all parts are complete.
2. Fill the nebuliser with the inhalation solution as per your doctor's instructions. Please ensure that you do not exceed the maximum level.
3. Please connect the nebuliser to the device using the air hose and then plug the device into the electricity supply (230V~ / 50Hz).
4. To start the treatment, switch the device on by pressing the On/Off switch to "I" and place the mouthpiece in the mouth or fit the mask.  
If possible use the mouthpiece as this gives a better drug delivery to the lungs. However, you may prefer to use the facemask and you will find two sizes are supplied to suit different size faces. The mask should completely enclose the mouth and nose areas.  
**ATTENTION:** this device is rated for intermittent use:  
30 minutes ON/30 minutes OFF.
5. During the therapy, breathe in and out calmly. Sit in a relaxed position with the upper part of the body upright. Stop inhalation if you feel unwell.
6. After completion of the inhalation period recommended by your doctor, please switch off the device (0) and unplug it from the electricity supply.
7. Empty the remaining medication from the nebuliser and clean the device as per the instructions in this manual.

## 3. Cleaning

---

**The nebuliser, face mask, and mouthpiece may only be used by one person.** After application, all parts must be taken apart and rinsed under flowing water. Then place all parts on a clean paper towel and leave to dry.

- **Never immerse components into hot water!**
- **The device must not come into contact with water or heat!**

The device and the air hose should be wiped out once a day with a clean, moist cloth – **the device must not be plugged into the electricity supply during this process!**

#### **4. Maintenance, care, and service**

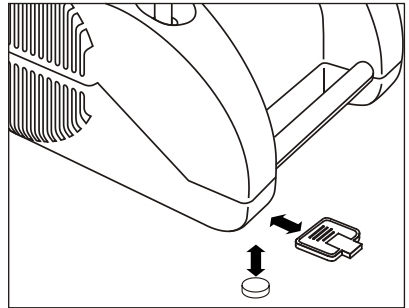
---

You can procure all necessary spare parts from your dealer or pharmacist, or please contact Microlife directly for details of your local Microlife supplier.

You will find detailed user information on our products and services at [www.microlife.com](http://www.microlife.com).

We recommend replacing the nebuliser, mouthpiece, and masks after 3 months use.

The filter should be continually checked for cleanliness and replaced if dirty, or after a maximum of 3 months use. Spare filters are provided with the device. To replace the filter, open the compartment on the bottom side of the device marked «Open» and exchange the filter as shown.

## 5. Malfunctions and actions to take

---

<b>Malfunction</b>	<b>Action</b>
The device cannot be switched on.	<ol style="list-style-type: none"><li>1. Check whether the device is correctly plugged into the electricity supply.</li><li>2. The device has switched off automatically due to overheating. Wait until it has cooled down.</li></ol>
The nebuliser only functions poorly or not at all.	<ol style="list-style-type: none"><li>1. Check whether the air hose is correctly connected at both ends.</li><li>2. Make sure that the air hose is not kinked or blocked.</li><li>3. Check whether the nebuliser is correctly assembled.</li><li>4. Make sure that medication has been added.</li></ol>

The service team at Microlife will be pleased to provide assistance if you have any questions or problems!

## 6. Additional safety instructions

---

- If there is visible damage to the device or power cord, please stop using the device.
- The device is only allowed to be used for the purpose described in the instructions. The manufacturer is not liable for damages due to incorrect usage.
- Do not subject the device to extreme temperatures, moisture, dust or direct sunlight.
- Never try to repair the device by yourself. Unauthorised opening of the device will render the guarantee void.
- Ensure that children do not use the device without supervision; some parts are so small that they could be swallowed and present a choking hazard.

- If the device has received a hard knock (fallen on the floor), you find a malfunction or damage, or notice something unusual, you should have the device checked by your national Microlife customer service organisation.

## 7. Guarantee

---

This device is **guaranteed for 2 years** from the date of purchase. The guarantee covers the device, not the replaceable components like the nebuliser, masks, mouthpiece, air hose, and filter. The guarantee is void if the device is improperly used or changes made by third parties. The guarantee is only valid on presentation of the bill or a guarantee card completed by the dealer or pharmacy you purchased it from.

## 8. Technical specifications

---

<b>Nebulisation rate:</b>	0.3 ml/min. (3 ml inhalation takes less than 10 minutes)
<b>Particle size:</b>	2.0 µm (MMAD with 0.9% NaCl using API system) or 5.0 µm (MMD with 0.9% NaCl using Malvern system)
<b>Compressor air flow:</b>	10 l/min.
<b>Acoustic noise level:</b>	53 dBA
<b>Power source:</b>	230 V~, 50 Hz
<b>Power cord length:</b>	2 m
<b>Operating temperature:</b>	10 - 40 °C
<b>Operating humidity:</b>	10 - 95% relative humidity maximum
<b>Storage temperature:</b>	-25 - +70 °C
<b>Storage humidity:</b>	10 - 95% relative humidity maximum
<b>Weight:</b>	1800 g
<b>Size:</b>	195 mm x 160 mm x 105 mm
<b>Reference to standards:</b>	EN60601-1, EN60601-1-2 (EMC)

Subject to technical change without notice!

### Electromagnetic Information

Portable and RF communication devices such as cellular phones, pagers, etc. can interrupt operation of electrical medical equipment. For this reason your compressor must be placed far enough away from these devices to prevent interferences. This device meets EN 60601-1-2 standard for electromagnetic compatibility (EMC).

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